


# FACE SHIELD

reusable  
with adjustable band

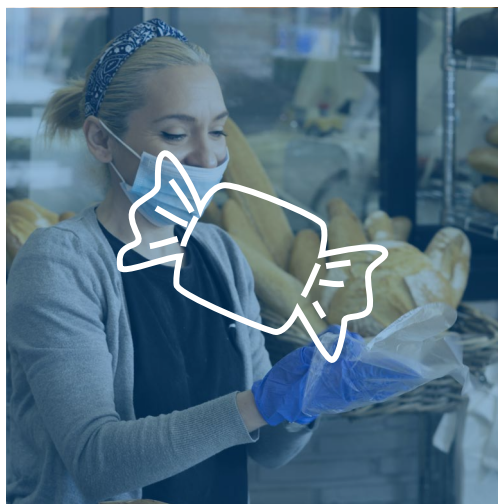

# Face shield with adjustable plastic band 300 x 220 x 0.3 mm

## Properties:



The surface suitable for disinfection with alcohol-based cleaning agents.



Transparent material, providing good visibility.



Adjustable band ensuring easier fitting to the head size.



Weight: 70 g.




Band: polyethylene (PE).  
Visor: transparent PET, 0.3 mm thick.



Self-assembly product.



## Quick and easy assembly:

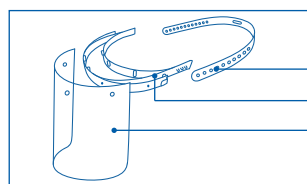


### Content of the kit:


1. Adjustable band
2. Head band
3. Visor




1 Before assembly of face shield make sure to wash your hands thoroughly and disinfect face shield parts.

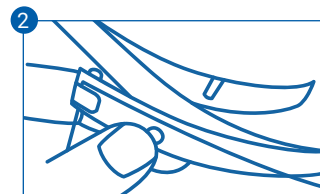

2 Push the transparent visor into mounting slots and inserts on the head part.

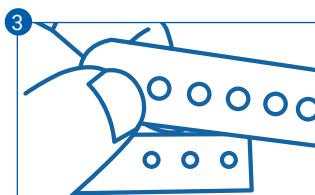

3 Push openings on the band into mounting inserts on the head part.

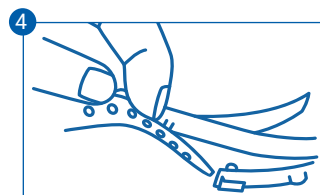

4 Adjust the band to the head size using openings on the band.

## Face shield with adjustable band

Transparent face shield, easy-to-use, provides face protection, reduces the risk of contact with airborne viruses and germs. Ensures effective protection of mouth, eye and nose areas. It will work perfectly as a protection against instinctive touching of the face with infected hands.

Face shield comprises transparent plastic PET visor and adjustable polyethylene mounting band. With adjustable band the face shield can be fitted to the head size easily. Appropriate design allows for wearing of glasses.

The face shield is comfortable to use, light-weight, and can be worn for many hours. Rounded from the bottom, improves work comfort.

The face shield is reusable. It is recommended to wear the face shield with a face mask.

## Packing:



20 pcs., 50 pcs., 100 pcs.



Tulplast sp. z o.o. sp. k.

ul. Wiosenna 27

62-021 Paczkowo

Phone no.: +48 61 815 70 16, +48 61 815 70 17

[www.tulplast.pl](http://www.tulplast.pl)

## Any questions?

### Customer service consultant

Piotr Borkowski

Sales specialist

Mobile no.: +48 604 909 051

Email address: [order@tulplast.pl](mailto:order@tulplast.pl)